Contour next

Blood Glucose Monitoring System

NO CODING

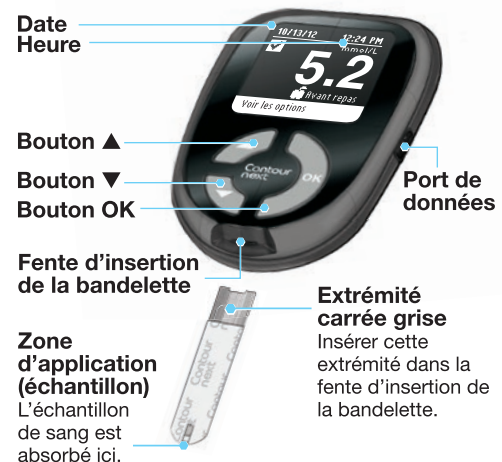
Your CONTOUR®NEXT Meter

Your CONTOUR®NEXT Test Strip

CAUTION: Use only CONTOUR NEXT test strips with your CONTOUR NEXT meter.

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Turning the Meter On or Off

- Press and hold the **OK** button until your meter turns On or Off.
- Inserting a test strip into the meter test strip port will turn the meter on.

Up/Down Scroll/OK Buttons

- The ▲ or ▼ scroll buttons allow you to change values or options one at a time. You can hold the ▲ or ▼ button to scroll.
- Press the **OK** button to select the highlighted option.
- To exit from the Logbook or Trends and return to the Main Menu, press the **OK** button.
- To exit from Setup and return to the Main Menu, scroll down to **Main Menu** and press the **OK** button.

Please ensure that you read the CONTOUR®NEXT user guide and lancing device insert for full instructions. For In Vitro Diagnostic Use

Initial Setup of Your Meter

You must set up your meter before doing your first blood sugar test.

1. Press and hold the **OK** button until the meter turns on.

2. Press the ▼ button if needed to highlight your desired language. Scroll down to see more languages. Press the **OK** button to select.

This screen example shows the language selections. The language order may differ depending on your country.

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3. A screen confirms your language.
4. Press the **OK** button to complete setup. Your meter is now ready to test your blood sugar.

You can also change your language, time and date by going to the **Main Menu** and selecting **Setup**. Please see the user guide for instructions.

For more information, go to www.diabetes.ascensia.com or contact Customer Service at: 1-800-268-7200.

ASCENSIA
Diabetes Care

Manufactured for:
Ascensia Diabetes Care Holdings AG
Peter Merian-Strasse 90
4052 Basel, Switzerland

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Blood Sugar Testing – Fingertip

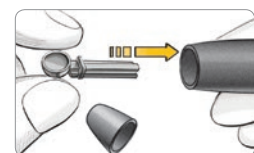
⚠️ WARNING

Always wash your hands well with soap and water and dry them well before and after testing, handling the meter, lancing device or test strips.

1. Remove the endcap from the lancing device.

2. Loosen the round protective cap on a lancet by rotating it ¼ turn, but do not remove it.


3. Insert the lancet firmly into the lancing device until it comes to a full stop.


4. Twist off the round protective lancet cap.

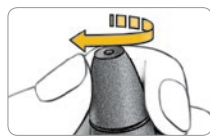5. Replace the endcap.

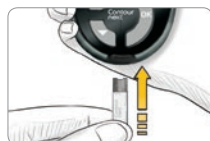6. Adjust the endcap dial to your preferred puncture depth setting.

7. Insert the square grey end of the test strip into test strip port until the meter turns on.

The meter will turn on and you will see the **Apply Blood** screen. The meter is now ready to test.

8. Press the lancing device firmly against the puncture site and press the release button.

Stroke your hand and finger towards the puncture site to form a drop of blood.

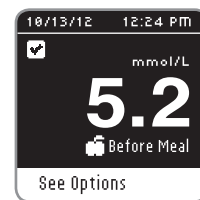
9. Immediately touch the tip of the test strip to the drop of blood.

If the first blood drop is not enough, the meter will beep twice. You have about 30 seconds to apply more blood to the same strip. Follow instructions on the meter screen.

10. The AutoLog* Screen appears. Select **No Mark, Fasting, Before Meal** or **After Meal**, and then press the **OK** button.

If your result is above or below your set Target ranges, you will always see your result without pushing a button.

After you have made your selection, your test result will appear. Your test is now complete and your test result is marked.

If your test result is outside your Target range, you will see your result with a notice and will not see the AutoLog screen.

11. To view Options for setting a reminder or adding a note, press the **OK** button before you remove the test strip.

Please see the user guide for instructions on how to use the Options. To turn your meter off, remove the test strip.

* Your meter comes with AutoLog on. See your user guide for information about the AutoLog feature.

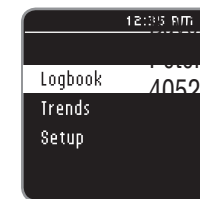
Your meter is preset and locked to display results in mmol/L. If your result is in mg/dL, contact Customer Service at: 1-800-268-7200.

CAUTION:

- The lancing device provided with your kit is not suitable for use by health care professionals. It must not be used on more than one person due to risk of infection.
- To remove the lancet from the lancing device refer to the detailed instructions in the lancing device insert.
- The lancing device graphics are for illustration purposes only.

The Main Menu Screen

1. **Logbook** will be highlighted. To view your logbook, press the **OK** button.

2. To view all of your averages and summaries, press the **▼** button to highlight **Trends** and press the **OK** button.

3. To change or adjust meter settings, highlight **Setup** and press the **OK** button.

Potential Biohazard

All blood glucose measuring systems are considered biohazardous.

All products or objects which come in contact with human blood, even after cleaning, should be handled as if capable of transmitting infectious diseases.

The user should follow the recommendations for prevention of blood-borne transmissible diseases in health care settings as recommended for potentially infectious human specimens.

See your User Guide for complete instructions.

Ascensia, the Ascensia Diabetes Care logo, Clinilog, Contour, Glucofacts, and the No Coding logo are trademarks and/or registered trademarks of Ascensia Diabetes Care Holdings AG. TM see www.ascensidiabetes.ca/en/tm-mc

For information on patents, and related licenses, see: www.patents.ascensia.com

© 2016 Ascensia Diabetes Care Holdings AG. All rights reserved.

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© 2016 Ascensia Diabetes Care Holdings AG. Tous droits réservés.